

VARIONET PRESBYOPIA TEST

Dioptric Power

First, **print this page** on your printer. **Do not use this chart directly** on your computer screen.

The printed chart **must be exactly 5 inches wide**. If your copy is not 5 inches wide, contact us through the [Varionet contact form](#) and we will mail you a free eyechart to help you determining your vision correction.

Once your chart copy is correctly sized, start reading it from a distance of **about 14 inches**. Your needed dioptric power comes with the first line you have difficulty reading.

When [ordering Varionet reading glasses in our Online Store](#), select this strength.

	DIOPTER
If this line is difficult to read, use	+1.00
If this line is difficult to read, use	+1.50
If this line is difficult to read, use	+2.00
If this line is difficult to read, use	+2.50
If this line is difficult to read, use	+3.00